

Thank you for purchasing the Littman Stethoscope (Cross bar style) Air tag holder See below for assembly:

Place the Back Piece (green below) over the cross bar tube arrow up on round design towards ear tips, it will be snug and can be flexed slightly to fit over the tube make sure it is centered.

Take the Middle piece (blue below) and align it over the cross tube note curved channel and align with back piece. Insert air tag or other tracker in top piece (red below) and align over the middle and back section using the two post tabs and holes in middle piece. Inserts screws and tighten (Phillips head or + screw driver

Any issues let us know shamrockwood315@gmail.com. www.Shamrockwood315.com
Youtube @yourcouchsuck

